

# kliniderm<sup>®</sup> superabsorbent

**Kliniderm superabsorbent** is a four-layer superabsorbent dressing held together by a hypoallergenic seal. **Kliniderm superabsorbent** is a protease modulator and is indicated for moderate to highly exuding chronic and acute wounds. Can be used effectively under compression<sup>1</sup>.

**Kliniderm superabsorbent** removes excessive MMPs and exudate, provides moist wound healing, prevents maceration and excoriation to the wound bed.

## Where can Kliniderm superabsorbent be used?

Moderate exudate	Granulating tissue	Under compression	✓	Dry wounds	✗
High exudate	Sloughy tissue			Low exudate	
			Necrotic tissue		

## Where exudate is moderate to high.

Diabetic foot ulcers	Surgical dehiscence	Infected wounds	✓	Small sinuses	✗
Pressure ulcers	Large cavity wounds	Burns		Small cavities	
Leg ulcers					

## Applying Kliniderm superabsorbent.

In the majority of instances **Kliniderm superabsorbent** may be applied directly to the wound bed as a primary dressing, thus ensuring optimised exudate handling capacity:

- Choose the appropriate dressing size which is slightly larger than the wound size
- Place directly to the wound bed as a primary dressing
- Only apply fixation to wound edges – try not to entirely occlude the Kliniderm dressing

## Ordering details

Size	Pieces per box	Product code	PIP code
20cm x 20cm	15	40511703	394-7157
20cm x 30cm	10	40511704	394-7165
10cm x 10cm	10	40511706	394-7132
10cm x 15cm	10	40511707	394-7124
10cm x 20cm	10	40511713	410-0087



[www.kliniderm.co.uk](http://www.kliniderm.co.uk)

For further information please contact your local representative:

**Caroline Taylor**

Mobile: +44 (0) 7531 707900 Email: [caroline.taylor@hrhealthcare.co.uk](mailto:caroline.taylor@hrhealthcare.co.uk)

**h&r healthcare**  
delivering innovation

AM100-1

<sup>1</sup>. Free swell absorption, 2015. Data on file.